

Immediate Mode

STEP-BY-STEP GUIDE - BREEZY BLUE® DISINFECTION



1. Clean

Follow your standard cleaning procedures to remove garbage, soil, dirt, and debris.



2. Prepare

Ensure no one is inside the room. Position Breezy Blue® in a corner, with the jets pointed towards the center of the room.



3. Fog

Using the Smart Controller, press the button for the current room size and exit the room during the seven-second countdown.



4. Contact Time

Close the door and place the door-handle warning sign. Allow at least 10 minutes of contact time before re-entry.



5. Re-entry

If immediate re-entry is needed, open the door to ventilate any remaining hydrogen peroxide.



For more guidance, visit
BreezyMed.com/Support
or scan the QR code.

