



Gen 2

Assembly Guide

Contacting Breezy Med

For product information, sales, service, and technical support:

Phone: (833) 273-3991

Email: info@breezymed.com

Product Overview

Box Contents

- Handle
- Buggy Base
- Wheel Casters (qty: 4)
- Smart Controller Holder
- Charging cable clips (qty: 4)

Required tools (not included)

- #2 Phillips screwdriver

Attach handle to Buggy Base

- Remove the two screws from the bottom legs of the handle and slide the handle into the two bosses on the top of the Buggy base. Insert the screws into the holes on the underside of the Buggy base opposite the handle, then tighten securely.



Attach caster wheels to the Buggy base

- Press the stems of the caster wheels into the four receptacles in the corners of the Buggy base by pressing firmly on the caster stem until they are fully inserted.
- The stem will not be visible when the caster wheels are fully inserted.



Install Smart Controller holder

- Remove the tape covering the two screws inside the holder. Attach the holder to the bracket on the Breezy Buggy handle and tighten the screws.
- To install the Smart Controller (not included), remove the two screws on top of the Smart Controller holder and remove the cap.



- Slide the Smart Controller into the holder and reassemble using the provided screws.



Install Smart Controller charge cable

- Plug the Smart Controller charge cable (included with smart controller) into one of the USB ports on the Buggy base and the USB port on the Smart Controller.



- Use the four cable clips to secure the cable along the side of the Breezy Buggy handle starting at the bottom.
- To install each clip, place the cable into the clip recess and snap the clip onto the handle. Install the clips in orientations and locations shown below.



Install Breezy Blue

- Plug the included short Breezy Buggy power cable into Breezy Blue while tipping Breezy Blue on its side as shown below. Set Breezy Blue into the Buggy base with the fogging jets facing forward. Wrap any excess cable around the handle of the Breezy Blue.



Additional Information

Further product information and the complete user manual can be found at <https://breezymed.com/support>